

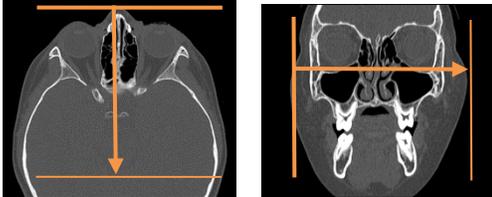
Application Examples: fracture

Scan Protocol (See for Technical Factors)

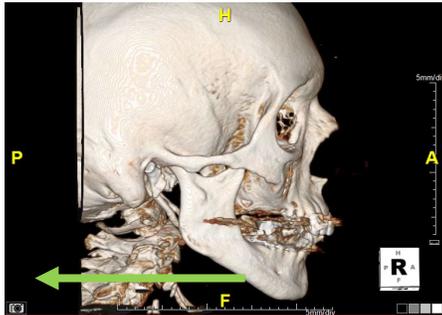
**Face****Reformations:** Coronal and sagittal MPRs and 3D VR spin and tumble.

1. *2x2 mm Coronal and sagittal MPRs.* If unable to place patient in ideal position, make a true axial MPR data set parallel to hard palate.

Series Description:

**COR****SAG**

2. *3D VR Spin.* Load thin data set into GL Bone workflow. Starting from a Right Lateral position, rotate volume 360 degrees to visualize face first with image number set at 36. Capture one at the end to close rotation. If hardware is present, repeat rotation using Hardware Bone template. Both spins in one series.

Series Description: **3D**Series Number: **1001**

3. *3D VR Tumble.* Starting from (H) or “top of head” view, tumble volume up 360 degrees with image number set at 36. If hardware is present, repeat rotation using Hardware Bone template. Both tumbles in one series.

Series Description: **3D**Series Number: **1002**